

MERIDIAN

TEMPERATURE SCREENING KIOSKS

Head Unit Attachment Instructions


GETTING STARTED

Upon arrival of the Personnel Management Kiosk, the head unit will be wrapped in the position shown in the picture shown right. These instructions will walk you through how to properly attach and secure the head unit in place to begin using your kiosk.




PREPARING THE MOUNTING STAND

After unwrapping the unit, unscrew the cap on top of the mounting stand. Carefully guide the cables into the stand, lowering the mounting joint into place.

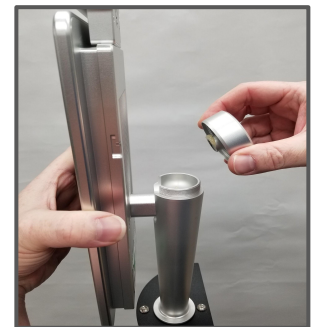

SECURE THE HEAD UNIT

Place the cap back on top of the stand and screw on to tighten. You may loosen the cap to adjust the head unit of the PMK to a position more suitable for your needs. Tighten the cap to secure the head unit.

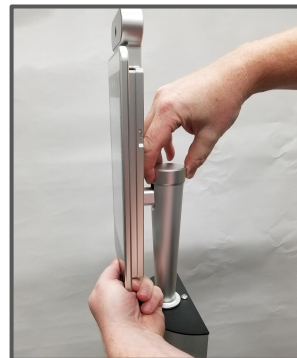

CORPORATE HEADQUARTERS
312 S Pine Street, Aberdeen, NC
sales@mzero.com
+1 866 454 6757

Component specifications subject to change without notice. Revision Date: January 14, 2021

This device is not FDA-cleared or approved. This temperature device should not be solely or primarily relied upon to diagnose or exclude a diagnosis of COVID-19, or any other disease or health condition. Elevated body temperature in the context of use should be confirmed with secondary evaluation methods, such as a non-contact infrared thermometer or clinical grade contact thermometer.

SOFTWARE DEVELOPMENT LAB
30 Eglinton Ave, Ste 808, Mississauga, Ontario, Canada

 **Complies with
UL/SCA C22.2
No. 62368-1**

E212745

